- I - SET PER TORACENTESI, PARACENTESI E ACCESSORI
- GB - SET FOR THORACENTESIS, PARACENTESIS AND ACCESSORIES
- F - KIT POUR THORACENTESE, PARACENTESE ET ACCESSOIRES
- D - THORAKOZENTESE-/PARAZENTESE-SCHLAUCHSYTEM UND ZUBEHÖ R
- E - JUEGO PARA TORACENTESIS, PARACENTESIS Y ACCESORIOS
- P - SET PARA TORACOCENTESE, PARACENTESE E ACESSÓRIOS
- N - SETT TIL THORACOCENTESE, PARACENTESE OG TILBEHØR
- NL - SET VOOR THORACOCENTESIS, PARACENTESIS EN ACCESSOIRES
- FIN - TORAKOSENTEESI-, PARASENTEESIPAKETTI JA LISÄVARUSTEET
- S - SET FÖR TORAKOCENTES, PARACENTES MED TILLBEHÖR
- DK - SÆT TIL THORACOCENTESE, PARACENTESE OG TILBEHØR
- PL - ZESTAW DO TORACENTEZY I PARACENTEZY Z AKCESORIAMI

I

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GB

APPLICATIONS

Aspiration and collection of waste fluids drawn from the patient. In detail, the models named TORASET and TORAMATIC are used in cases of pleuric effusion and therapeutic thoracentesis in order to reduce the symptoms caused by pleuric effusion, while the sets named PARASET have been specifically designed for drainage of the ascitic fluid during execution of paracentesis methods.

INSTRUCTIONS FOR USE

- Locate the point of pleuric effusion (for thoracentesis) or the cavity to be treated using standard diagnostic methods such as percussion, radiology or ultrasonography.
- The point where to introduce the needle should be selected so as to avoid the vascular and nervous structures in the intercostal space close to the lower margin of the ribs (for thoracentesis techniques) or under the skin (for paracentesis techniques).
- Disinfect the localised skin area.
- A local anaesthetic can be injected under the skin.

TORASET AND PARASET MODELS

- Thoroughly wash the hands and open the packet.
- Put on a pair of sterile gloves (not provided).
- Prepare a sterile field on which to place the components contained in the set.
- Remove the protective cap from the needle.
- Introduce the needle ensuring that the desired cavity has been reached. If it does not penetrate to the due depth, it is advisable to remove the needle and recheck the point of effusion with diagnostic methods (when using cannula needles, the internal metallic cannula must be removed after insertion).
- Connect the needle to the extension and the syringe to the three-way tap.
- Position the tap so that the aspiration flow is from needle to syringe, using the convenient indications embossed on the tap.
- Proceed with aspiration
- When the syringe is full, orient the tap in such a way that the flow is from syringe to bag, then drain the fluid which will flow directly into the collection bag.
- If necessary, repeat the aspiration operation several times.

TORAMATIC MODEL (fitted with single-acting valves)

- Thoroughly wash the hands and open the packet.
- Put on a pair of sterile gloves (not provided).
- Prepare a sterile field on which to place the components contained in the set.
- Remove the protective cap from the needle.
- Introduce the needle and ensure that the desired cavity has been reached. If it does not penetrate to the due depth, it is advisable to remove the needle and recheck the point of effusion with diagnostic methods.
- Connect the needle to the extension and proceed with aspiration.
- When the syringe is full, empty it. The fluid will flow directly into the collection bag thanks to a single-acting valve system.
- If necessary, repeat the aspiration operation several times.

TORASET-VER, TORAMATIC-VER and PARASET-VER MODELS (with Veress needle)

- Thoroughly wash the hands and open the packet.
- Put on a pair of sterile gloves (not provided).
- Prepare a sterile field on which to place the components contained in the set.
- Remove the protective cap from the needle.
- If necessary, set the desired penetration depth with the stopper on the external cannula of the needle (the notches are spaced every 1 cm).
- Introduce the needle in the desired cavity taking care that the internal cannula draws back completely, checking positioning by means of the red indicator identifiable from the transparent connector. When the red indicator returns to the initial position, the internal cannula has been released and is in the operating position (in the cavity).
- Connect the syringe to the three-way tap (for Toraset and Paraset models); for the Toramatic models the syringe is already connected.
- Proceed with aspiration following the instructions for the two models TORASET-PARASET and TORAMATIC.
- If necessary, repeat the aspiration operation several times.

SPARE BAG FOR TORASET and PARASET MODELS

- Position the tap so that the aspiration flow is from needle to syringe, using the convenient indications embossed on the tap.
- Disconnect the bag being used from the tap through the Luer connection.
- Connect the spare bag in the same position and continue with the aspiration procedure following the instructions for the two models TORASET and PARASET.

WARNINGS

- Usage allowed only to qualified physicians. BIOSERVICE declines every responsibility for damages caused to the patient when the device is used in an unproper way, with wrong manoeuvres or if handled by non qualified personnel.
- During handling of the device, it is recommended to use an aseptic technique and to prepare the skin properly in order to reduce the risks of infection.
- Always check proper connection of the needle and the syringe.
- Do not use the device for permanent drainage. Remove as soon as the therapeutic thoracentesis or paracentesis procedure has been completed.
- Do not use the device for infusion.
- Do not reinfuse the aspirated fluid for any reason whatsoever.
- Do not attempt to fill the bag over its maximum capacity.
- Ensure that the tube is positioned so as to allow regular fluid flow.
- Do not place the bag on the floor, since this is a certain source of infection and malfunctioning of the device.
- E.T.O. sterilized. Do not re-sterilize.
- Sterility is guaranteed if the packaging is undamaged.
- Disposable single-use/single-patient: re-use of the disposable may cause cross infection to patient and/or user.
- Pyrogen-free
- Expiry: 5 years with undamaged packaging and if stored in normal storage conditions (keep in a fresh and dry place, avoid the exposure to light and high temperature).
- To safeguard personnel, when required by hospital regulations or by the department, it might be necessary to wear sterile gloves, masks and protective glasses (Warning: these accessories are not supplied with the device).
- Recapping of the needles must be considered a risky operation. Percutaneous puncture with contaminated needles may cause grave diseases such as hepatitis, AIDS or other infectious diseases. In the event of puncture with a contaminated needle, immediately report the event to a health worker and observe the established procedures.
- Discard after use in a container for sharp objects, leakproof and resistant to piercing.
- Dispose of the device in conformity with the legislation in force.
- Do not use devices for premature infants, newborns And children under 10 kg.

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