- I - CANNULA PER ASPIRAZIONE CHIRURGICA SINGOLA
CANNULA E SET PER ASPIRAZIONE CHIRURGICA
- GB - SINGLE SURGICAL ASPIRATION CANNULA
CANNULA AND SURGICAL ASPIRATION SET
- F - CANULE POUR ASPIRATION CHIRURGICALE SIMPLE
CANULE ET KIT POUR ASPIRATION CHIRURGICAL
- D - EINZELNE CHIRURGISCHE SAUGKANÜLE
KANÜLE UND SCHLAUCHSYSTEM FÜR CHIRURGISCHE ANSAUGUNG
- E - CÁNULA DE ASPIRACIÓN QUIRÚRGICA SIMPLE
CÁNULA Y JUEGO PARA ASPIRACIÓN QUIRÚRGICA
- P - CÂNULA DE ASPIRAÇÃO CIRÚRGICA
CÂNULA E SET DE ASPIRAÇÃO CIRÚRGICA
- N - SÆRSKILT KIRURGISK OPPSUGINGSKANYLE
KIRURGISK OPPSUGINGSKANYLE OG SETT
- NL - CHIRURGISCHE AFZUIGCANULE
CHIRURGISCHE AFZUIGCANULE EN -SET
- FIN - YKSITTÄINEN KIRURGINEN IMUKANYYLI
KIRURGINEN IMUKANYYLI JA -PAKETTI
- S - ENKEL KANYL FÖR KIRURGISK ASPIRATION
KANYL MED SET FÖR KIRURGISK ASPIRATION
- DK - SÆRSKILT KIRURGISK ASPIRATIONSKANYLE
KIRURGISK ASPIRATIONSKANYLE OG -SÆT
- PL - KANIULA ASPIRACYJNA POJEDYNCZO LUB W ZESTAWIE

8450330000 rev.03 du 29/09/2016

I

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GB

APPLICATIONS

Intraoperative aspiration of waste fluids and blood and when required localised aspiration.

INSTRUCTIONS FOR USE

- The device is packed in a double package which assures aseptic handling of the device.
- The operator will open the first packet keeping the product in the second packet until the time of use.
- Extract the device and securely connect it to the suction tube (for single cannulas) or to the aspirator (for cannula + set).
- Perform the aspiration operation.
- In the models fitted with a vacuum control device, intermittent aspiration can be carried out, occluding the hole on the top of the grip with the thumb.

WARNINGS:

- Usage allowed only to qualified physicians. BIOSERVICE declines every responsibility for damages caused to the patient when the device is used in an improper way, with wrong manoeuvres or if handled by non qualified personnel.
 - The device must be used by qualified medical/nursing staff.
 - During handling of the device always use a sterile procedure in order to reduce the risks of infection.
 - Always ensure that the connections are perfectly sealed and that the tube is well extended so as to allow regular flow.
 - Never place the device on NON-STERILE surfaces since these are certain sources of infection.
 - Do not use the device for infusion.
 - Disposable single-use/single-patient: re-use of the disposable may cause cross infection to patient and/or user.
 - Pyrogen-free.
 - Expiry: 5 years with undamaged packaging and if stored in normal storage conditions (keep in a fresh and dry place, avoid the exposure to light and high temperature).
 - EtO sterilized. Do not re-sterilize.
 - Sterility is guaranteed if the packaging is undamaged.
 - To safeguard personnel, when required by hospital regulations or by the department, it might be necessary to wear sterile gloves, masks and protective glasses (Warning: these accessories are not supplied with the device).
- Dispose of after use in conformity with the legislation in force.
- Do not use devices for premature infants, newborns And children under 10 kg.

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